1

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1

Download and install the ECG247 app on your mobile phone

2

Open the app and follow the instructions

3


Enter your mobile number

Get your single use code on SMS

4


Place the sensor on the electrode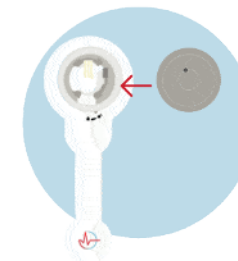

Rotate to the right

5


Connect the sensor to the phone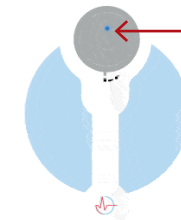

Press the button in the center of the sensor until the blue light comes on

6

Prepare for test

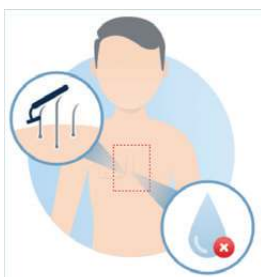Wash and dry the skin. Hair should be removed.

7

Attach the sensor to the center of the chest

Remove the paper from the adhesive side

8

The heart rhythm is recorded and analyzed

You can now use the phone as usual, but be sure to have it near the sensor during the test period

9

Use the sensor for at least 3 days. The test is stopped by pressing «Stop test»

Stop test

10

Order a review of the test results from our heart specialists or share the results with your doctor

Order new sensors and electrodes at ECG247.com

PRIVACY Your privacy is important to us. We protect your data and do not share your information. Read more at ecg247.com

RESPONSIBILITY The heart signals have different shapes and sizes in different people. We cannot guarantee that this test finds all different rhythm disorders in everyone. Use of sensor and associated software is therefore at your own risk. The sensor cannot look for signs of myocardial infarction. Any rhythm disturbances must always be confirmed by a doctor. Read more at ecg247.com

Always contact a physician if you have symptoms from your heart.

WARNING The sensor should not be used in patients with pacemaker/ICD, children <10 kg and in situations requiring external defibrillator use. If the use causes itching or rash, the sensor should be removed. Do not swallow. The sensor must be kept out of reach of children. The battery cannot be charged.

ENVIRONMENT The sensor can be used many times - take good care of it.

You can shower with the sensor, but avoid bathing.

Complete instructions at ECG247.com

Made in Norway
AppSens AS, 4790 Lillesand
post@appsens.no